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Príručka

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COMPRESSOR NEBULIZER

Guidebook

Please read this guidebook carefully before operating this unit.

Your Nebulizer is intended for use in the treatment of asthma, COPD and other respiratory ailments in which an aerosolized medication is required during therapy. Please consult with your physician and/or pharmacist to determine if your prescription medication is approved for use with this Nebulizer. This Nebulizer is intended for single patient use.

English

Operating Cautions:

- Connect this product to an appropriate voltage outlet for your model. Do not run this product unattended.
- Never operate if this unit has a damaged cord or plug, if it has been dropped, or in any way, submerged into water, if it does not work properly. Return it to a service center for repair.
- If any abnormality occurs, discontinue use immediately until the unit has been examined and repaired.
- Always unplug the product immediately after use.
- Never block the air openings of the main unit or place it where the air openings may be obstructed.

Storage Cautions:

- Do not store the unit under direct sunlight, high temperature or humidity. Keep the unit out of reach of small children.
- Always keep the unit unplugged while not in use.

Cleaning Cautions:

- Do not immerse the unit in water. It may damage the unit.
- Disconnect the unit from the electrical outlet before cleaning.
- Clean all necessary parts after each use as instructed in this guidebook.

OPERATING YOUR COMPRESSOR NEBULIZER

Note: Prior to initial operation, the nebulizershould be thoroughly cleaned. Place your Compressor Nebulizer on a flat and stable surface. Be sure that you can easily reach the controls when you are seated. Gently twist the top part ofthe nebulizer counter-clockwise to disassemble the nebulizer. Fill the bottom section of the nebulizer with the medication prescribed by your physician. Be sure that the cone is put inside the bottom section. Gently twist the top part clockwise to reassemble the nebulizer. Be sure the two sections fit well. Attach one end of air tube to the base ofthe nebulizer. Attach the other end of air tube to the tube connector located on the front ofthe compressor. Attach mouthpiece or mask per your choice to the top section of nebulizer. Plug the power cord into an appropriate electrical outlet. Make sure at this stage, the power switch is at "Off" status. Press power switch to begin your prescribed treatment.

Place cone inside and fill medication

INDEX

1. Introduction -----	Page2
2. Product Identification-----	Page3
3. Important Safeguards -----	Page 3
4. Operating Your Compressor Nebulizer -----	Page 4
5. Cleaning -----	Page 5
6. Replacement Parts & Optional Accessories -----	Page 6
7. Specifications -----	Page 6

INTRODUCTION

Thank you for purchasing the Compressor Nebulizer. It is a compact medical device designed to efficiently deliver physician prescribed medication to the bronchial lung passages. With proper care and use, it will provide you with many years of reliable treatment.

This product is developed for the successful treatment of asthma, allergies and other respiratory disorders. It creates a stream of air that travels through clear tube to the nebulizer. When air enters the nebulizer, it will convert the prescribed medication into aerosol mist for easy inhalation.

Your Compressor Nebulizer should be used underthe supervision of a licensed physician and/or a respiratory therapist. We encourage you to thoroughly read this guidebook to learn about the features of this product. Any use of this product other than its intended use should always be avoided.

Important:

- The compressor motor has a thermal protector, which will shut off the unit before the unit is overheated. When the thermal protector shuts the unit off, please:
- + Switch off the unit.
 - + Unplug the unit from the electrical outlet.
 - + Wait 30 minutes for the motor to cool down before another treatment. Make sure the air openings are not obstructed.
- When treatment is finished, shut off the unit and unplug it from the electrical outlet.

CLEANING

It is recommended that the nebulizer, mouthpiece and mask be thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment ofthe day. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

Rinsing (after each treatment):

1. Disconnect the air tube, nebulizer, mouthpiece and mask.
2. Gently twist the nebulizer to open it.
3. Rinse the nebulizer, mouthpiece and mask with water.
4. Dry them with clean soft towel or let it air dry.
5. Reassemble the nebulizer when completely dry and put these parts in a dry, sealed container.

Disinfections:

Please follow the following steps to disinfect your nebulizer unless otherwise specified by your physician. It is suggested that the unit is disinfected after the last treatment of the day.

- Use one part white vinegar with three parts distilled water. Make sure mixed Ssolution is enough to submerge the nebulizer, mouthpiece, and mask.
- Complete Rinsing Steps 1-3.
- Wash nebulizer, mouthpiece and mask in warm water and a mild detergent. Then wash them in hot tap water.
- Submerge these parts in the vinegar and water solution for thirty minutes. Complete Rinsing Steps 3-5.

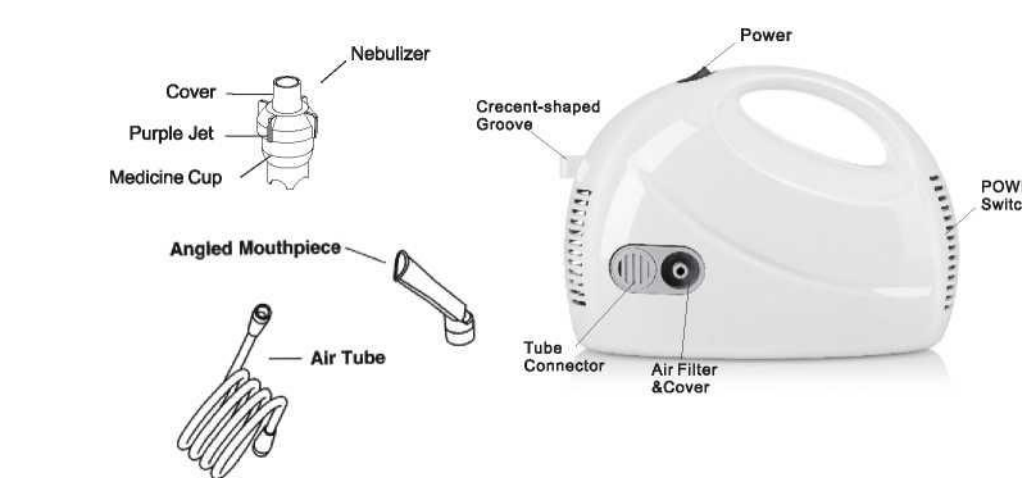
Cleaning the compressor:

- Wipe daily with a damp cloth.
- Do not use any powdered cleaners or soap pads, which may damage the finish.

Filter change:

- Do not use cotton or any other materials. Do not wash or clean the filter. Only use filters supplied by your distributor. And do not operate without a filter.
- Change the filter every 30 days or when the filter turns gray.
- Changing procedure:
 - + Remove the filter cover.
 - + Replace the used filter with a new one.
 - + Replace the filter cover.

PRODUCT IDENTIFICATION

IMPORTANT SAFEGUARDS

Note: Read all instructions carefully before use. The following basic precautions are needed when using an electrical product: **Caution:** Failure to read and observe all precautions could result in personal injury or equipment damage.

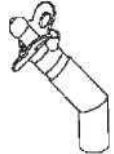
Product cautions:

1. To avoid electrical shock: Keep the unit away from water Do not immerse the power cord or the unit in liquid Do not use while bathing Do not reach for a unit that has fallen into water immediately unplug the unit.
2. Never operate the unit if it has any damaged parts (including power cord), if it has been dropped or submersed in water. Promptly send it to a service center for examination and repair.
3. The unit should not be used where flammable gas, oxygen or aerosol spray products are being used.
4. Keep the airvents open. Do not place the uniton a soft surface where the openings can be blocked.
5. Ifthe medicine cup is empty, do not attempt to operatethe unit.
6. If any abnormality occurs, discontinue use until the unit has been examined and repaired.
7. The unitshould not be left unattended while plugged in.
8. Do not tilt or shake the unit when in operation.
9. Disconnect the unit from the electrical outlet before cleaning, 山吨 and after each use.
10. Do not use other attachments unless recommended by manufacturer.

REPLACEMENT PARTS & OPTIONAL ACCESSORIES

- Replacement parts are available through your local retailer or they can be ordered byemail:
- Optional accessories include child mask, nose piece and baby pacifier inhaler.
- The pacifier inhaler allows baby to receive aerosol treatment easily through normal breath.

Baby pacifier inhaler

SPECIFICATIONS

Voltage	AC 220V/50Hz
Power	168VA
Medication Capacity	8 ml
Particle Size	0.5-10pm
Sound Level	<65dB(A)(1 meter away)
Average Nebulization Rate	>0.25mL/min (0.9% saline water)
Compressor Pressure Range	22-58psi (150-400 kPa)
Operating Pressure Range	9.5-19psi (65-130kPa)
Liter Flow Range	26.7L/Min
Operating Temperature Range	5°C~40°C(41°F~104°F)
Operating Humidity Range	15~80%RH
Storage Temperature Range	-25°C~-60°C(-13°Fto140°F)
Dimension	185x169x107mm
Weight	1.3 kg
Standard Accessories	Nebulizer, AirTube, Adult Mask,Child's Mask Mouthpiece, Filters (5)
Use life	5 years

Protection against electric shock: -Class II Equipment

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