# INSTRUCTION MANUAL

## ELECTRONIC BABY SCALE

DEAR CUSTOMER

THANK YOU FOR CHOOSING THIS BABY SCALE. PLEASE READ THESE INSTRUCTIONS FOR USE CAREFULLY AND KEEP THEM FOR FUTURE USE.

### SPECIFICATIONS


- Accurately measures weight in 5g increments up to 20kg
- HOLD feature captures precise weight measurement of an active baby
- TARE feature subtracts added weight of diaper, clothing or blanket
- Easy-to-read, white backlit LCD display
- Unique touch button design
- Includes 2xAAA batteries
- Auto power off and low battery indication

### DISPLAY

### POWER SUPPLY

Please insert batteries or remove the insulating foil from the batteries before using. When “Lo” appears in the display, the batteries need to be replaced.

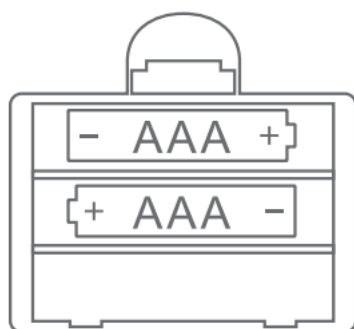### OPERATING INSTRUCTIONS

The weight is displayed in steps of 5g from a minimum weight of 30g. Press the button “”

When "0.000kg" appears on the display, the scale is ready for measurement. Place the baby on the scale carefully. Take particular care with the baby's head.

#### **HOLD FUNCTION:**

The scale has a "HOLD" function: AUTO HOLD, MECHANICAL HOLD.

AUTO HOLD: When the baby is placed on platform, he/she may move around. When he/she is still

enough, the AUTO HOLD function will establish the weight on display. The sign "" flashes and then weight is established.

MECHANICAL HOLD: If the baby is moving around too vigorously, you may select the MECHANICAL

HOLD function by pressing "" button. The sign "" flashes and weight is

established on display. The sign "" remains on display until you press "" button or "TARE" button. Then it shows "0.000kg" which means the scale is ready for next weighing.

#### **TARE FUNCTION:**

If you prefer to place a towel or blanket on the scale and weigh the baby, please do so BEFORE switching the scale on. If the towel or blanket weighs over 30g, please press TARE button.

"TARE" is shown on display. Then place the baby onto towel or blanket. It shows baby's weight.

#### **UNITS:**

The scale is with units of kilograms(kg) and pounds(lb), which can be switched by pressing "UNIT" button.

#### **SWITCH OFF**

After weighing is finished, you can switch off the scale by pressing "" button. Or, the scale will automatically switch off if no weighing is carried out within 180 seconds.

#### **OVERLOAD INDICATION**

The scale can accept a maximum load of 20 kg. When overloaded, "EEEE" sign will appear on display.

#### **CARE AND ATTENTION**

- Place the scale on a hard, flat surface to ensure greatest accuracy.
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid if necessary. Never immerse the scale in water or rinse it under running water.
- Protect the scale against knocks, damp, dust, chemicals, drastic changes in temperature, electromagnetic field and source of heat that are too near.
- Keep packaging material away from children.

- DO NOT dispose of the old batteries in fire. The batteries may explode or leak. Recycle or manage them as hazardous waste.
- If the scale is not used in a long period, please take the batteries out

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